

The GS Testing Module for function tests of external cardiac pacemakers

- ☑ transthroracic and intracardial pacemakers
- ☑ single- and dual-circuit cardiac pacemakers

- ☑ graphical display of the pace impulses





Technical Data

Load resistance: 500 or 1000 Ohm \pm 1%

Measuring ranges: $0 - \pm 20 \text{ V} \pm 0,1 \text{ V} \text{ or } \pm 5\% \text{ of measurement value} 0 - \pm 20 \text{ mA} \text{ at} 1000 \text{ Ohm} 0 - \pm 40 \text{ mA} \text{ at} 500 \text{ Ohm}$

0 - \pm 250 V \pm 0,1 V or \pm 5% of measurement value 0 - \pm 250 mA at 1000 Ohm 0 - \pm 500 mA at 500 Ohm

Maximum time resolution: $dt=20 \mu s$

Frequency measurement: 30 - 800 BPM, \pm 1 BPM or \pm 1% of measurement value Pulse rate: 30 - 800 BPM, \pm 1 BPM or \pm 1% of measurement value

Synchronization signals: sinus, square sinus, triangle, rectangle, trapeze, ISO, ventricular fibrillations (VF),

ventricular tachykardie (VT), line frequency, QRS

Pulse duration: 0,1 - 200 ms \pm 0,1 ms or \pm 5% of measurement value

Description of functions:

The GS Testing Module PACE serves for functional testing of external single- and dualcircuit cardiac pacemakers for intracardial or transthoracic stimulation with output asynchronous or demand pulses. **PACE** determines the pulse amplitude and the AV delay time. With the aid of a program-controlled test signal, the system can also automatically determine the refractory periods, the sensitivity (demand amplitude). and the frequencies. The internal load resistance can be set by program control to 500 Ohm or 1000 Ohm.

Determination of the pulse duration and the pulse amplitude:

The pulse duration T_P is measured as the time interval between the 50% signalcurve values, with reference to the maximum voltage. The system calculates and displays the pulse amplitude as the arithmetic mean of the pulse signal within the time intervall T_P .

inhibition amplitude and demand amplitude:

Once you start the automatic measurement function of inhibition amplitude, the selected test signal is increased per every pulse. Once you start the automatic measurement function of demand amplitude, the selected test signal is reduced per every pulse.

Demand (inhibition) frequency:


The demand (inhibition) frequency is the cardiac / ECG signal frequency, the pacemaker takes over stimulation of the heart. Once the ECG signal frequency falls below the frequency set at the pacemaker, the pacemaker will emit stimulation pulses. The PACE module generates test pulses at its input/output, beginning with a pulse rate of 70 ppm and then slowly falling until the pacemakers begins stimulation.

Output of test-signal pulses:


The output of simulated ECG pulses (test-signal pulses) takes place under program control, according to the parameters which have been entered in the ACTIMED Testing Program. The user can select among ISO, sine², triangular and square-waves pulses. Pulse durations are output within the range of 15 ... 200 ms.

(The specified measuring accuracy refers to the measuring element. Technical modifications and errors reserved. 03/2012)

Technical Data

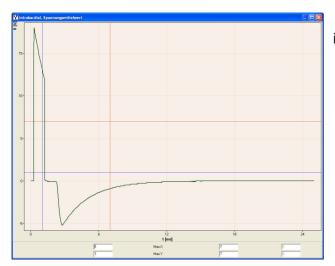

intracardial chart

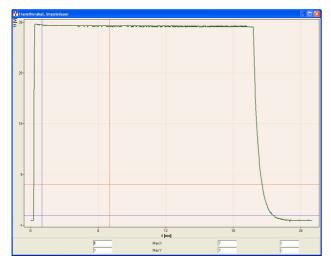

transthroracic chart,

(The specified measuring accuracy refers to the measuring element. Technical modifications and errors reserved. 03/2012)