# Function Tester STIMU-600



# Testing system for function tests of electro-stimulation equipment in accordance to IEC 60601-2-10

- touch screen or PC operation  $\overline{\mathbf{V}}$
- graphical display of signals  $\overline{\mathbf{A}}$
- measurement of effective current and effective voltage  $\square$
- measurement of interference current values  $\square$
- identification of beats  $\mathbf{\Lambda}$
- $\overline{\mathbf{A}}$ user specific language setting
- accumulator operarated handily device  $\overline{\mathbf{V}}$

Test and measurement technic for medicine and industry





# **Technical Data**

Supported by: Federal Ministry for Economic Affairs and Energy

on the basis of a decision by the German Bundestag

Mains Voltage: +5 V USB

or internal accumulator operation Protection class: internal power supply

Environmental temperature: Storage temperature:

+ 5 - + 40 °C -10 - + 50 °C

Timer:

OFF, 10, 20, 30 min, ± 1 sec

Interface:

1 x Mini USB Type B for PC connection 6 sockets 4 mm

Testing device connections:

4,3" TFT-Display Touchpanel

Mechanical data:

light weight metal case IP20 167 x 190 x 37 mm (W x H x D)

Dimensions: Weight:

Display:

Operation:

approx. 1,2 kg

Selectable languages:

german, english, french, polish spanish, italian, portuguese, turkish Load resistance: Ranges of measurement:

 $\pm$  0,1 V or 1 % of measurement value  $\pm$  10 V + 100 V  $\pm$  700 V 0 - 10 mA0 - 70 mA

10000 Ohm  $\pm$  3 %  $0 - 1 \, \text{mA}$ 2000 Ohm ± 1 %  $0 - 5 \, \text{mA}$ 0 - 50 mA0 - 350 mA1000 Ohm  $\pm$  1 % 0 - 10 mA0 - 100 mA 0 - 700 mA500 Ohm ± 1 % 0 – 20 mA 0-200 mA0 - 1400 mA 300 Ohm  $\pm$  2 % 0 – 33 mA 0 - 333 mA 0 - 2333 mA

0 - 1250 mA

Frequency bandwith for rms-value determination: 30 kHz Determination of rms-value for input signals: Measurement of maximum-value:  $\pm$  1 %

0 - 125 mA

Measurement of the DC component:  $\pm$  5 % Measurement of the interference-current signal: ± 5 % Frequency determination of sine-wave signals:

0,5 Hz - 10 kHz  $\pm$  0,1 Hz or  $\pm$  1 % of measurement value

0 - 8750 mA

# **Description of functions:**

The STIMU-600 serves for functional testing of electro-therapy equipment which operates with DC, low frequency, medium frequency or interference current. STIMU-600 checks the rms values for current and voltage, the beat for interference-current equipment, DC component and frequency.

The signal curves can be displayed. The measured values are displayed and can be stored in measurement protocols. The STIMU-600 determines the following measured values:

- rms current and rms voltage (maximum, rms value, plot of curve)
- displays values of the input signals
- frequency of the input signal and the rms signal
- DC component
- interference current
- beat frequency

STIMU-600 offers three measurement channels program-controlled, adjustable resistance. 300 Ohm, 500 Ohm, 1000 Ohm, 2000 Ohm and 10 kOhm can be set. In addition 80 ohms can be realized by an external resistor.

### Measurement of rms current and rms voltage:

Measurement of the rms values of voltage and current takes place by means of an rms-value converter. This unit converts either the signal from the selected channels or the sum of these signals into a true rms value.

### Measurement of frequency:

Determination of the signal frequency can take place for the original signals as well as for rmsvalue signals- or for the interference signals.

#### DC component:

80 Ohm ± 2 % (extern)

Measurement of the DC component takes place by means of low-pass filtering with a time constant of 400 ms.

## Measurement of interference current and determination of beat:

Interference-current devices have two or more electro-stimulation channels, one of it usually has fixed amplitude and frequency - and the other one, fixed amplitude and changeable frequency or phase. As described above, the current and frequency values for the individual channels can be measured for each channel. Additionally, the interference current measured as rms value of the sum of two individual channels and can be displayed.

(The specified measuring accuracy refers to the measuring element. Technical modifications and errors reserved. 10/2013)